



INSTRUCTIONS FOR USE

Pot suitable for storage of femoral head, tibial plateau or tendon.

Storage before pot is used:

- Store in clean, dry conditions away from direct sunlight

Operational Information:

- Always completely disengage cap threads before autoclaving.
- Autoclave at 121°C, 15 psig for 20 minutes before use.
- Suitable for a single dose of Gamma irradiation at between 25-35kGy
- Pots and caps can be stored as low as -80° C for up to five years.
- Frozen plastic is very brittle. Handle with care and avoid shocks or pressure when frozen.
- If to be excessively handled allow to return to room temperature first.
- Do not reuse due to risk of cross-infection.

